



SCHWERT
SWORD · ESPADA
EPEE · SPADA

SAFE-EX 2
patented

1033-61 / 13051 Aufbereitung und Pflege

Pflege und Aufbereitung vom SAFE-EX 2 Klingenhalter



DEMONTAGE:

Der SAFE-EX2 Klingenhalter ist vor der Reinigung, wie abgebildet, zu demontieren.

Achtung: Feder nicht entfernen.

REINIGUNG:

Bei allen Reinigungsvorgängen ist stets unsere „Anleitung zur Wiederaufbereitung von resterilisierbaren Handinstrumenten und deren Zubehör nach DIN EN ISO 17664“, in der jeweils gültigen Fassung zu beachten.


Vorreinigung: Bei starker Verschmutzung Reinigungsbürsten mit Nylonborsten verwenden.

Reinigung: Manuell oder maschinell – **Achtung:** Im Thermodesinfektor senkrecht stehend, mit Gewinde nach unten reinigen.

Spülung: Möglichst deionisiertes Wasser verwenden, um Wasserflecken zu vermeiden.

Trocknung: Mit Druckluft oder an der Luft (manuelle Reinigung).

WARTUNG:

Mit Medizinischem Weißöl, wie gekennzeichnet , ölen.

MONTAGE:

Die Montage erfolgt in umgekehrter Reihenfolge der Demontage.




3297 50 ml


Paraffinum Perliquidum
(Paraffin Öl / Medizinisches
Weißöl)

Zur Pflege der Instrumente
mit Gelenk und zur Benutzung
als Schleiföl. Physiologisch
unbedenklich nach DAB 10.


1 Das Endstück gegen den Uhrzeigersinn aufdrehen

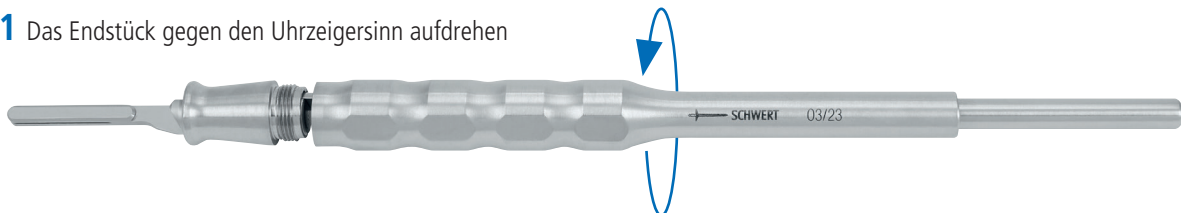

2 Entfernen des Endstücks und des Innenteils

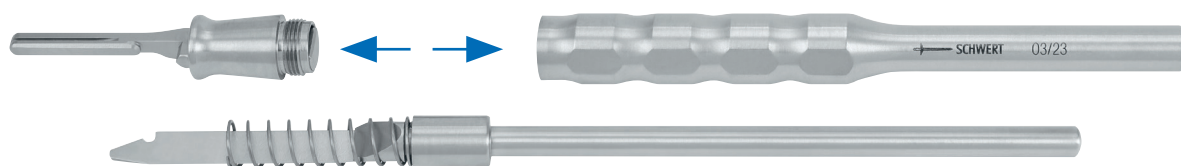

3 Nach der Reinigung mit Medizinischem Weißöl ölen

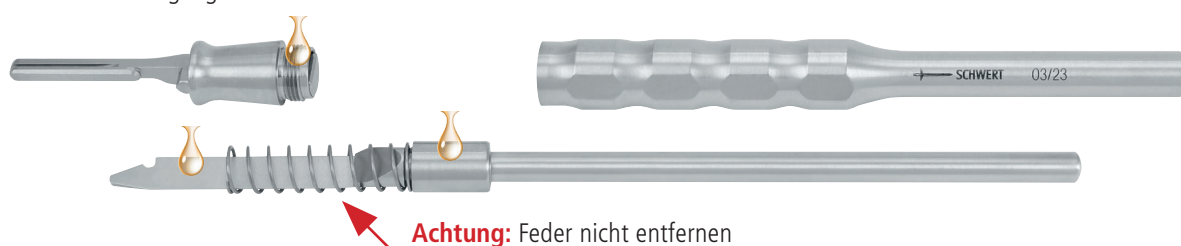



SAFE-EX 2 patented

1033-61 / 13051 Care and Maintenance

Maintenance and re-processing of SAFE-EX 2 Blade-Holder

DISMANTLING:

The SAFE-EX 2 Blade Holder has to be dismantled before cleaning, as shown.

Attention: Don't remove the spring.

CLEANING:

Please consider our "Application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664", in the currently valid version.


Pre-Treatment: Use cleaning brush with nylon bristles when incrustations are heavy.

Cleaning: Manual or mechanical – **Attention:** Cleaning in an thermal disinfectant: standing vertically, thread downwards.

Rinsing: Please use deionized water to avoid water spots.

Drying: Use compressed air or air-dry (manual preparation).

MAINTENANCE:

Oiling with Medical White Oil, as shown .

ASSEMBLY:

Assembly is made in reverse order to the dismantling.




3297 50 ml


Paraffinum Perliquidum
(Paraffin Oil / Medical White Oil)

For maintenance of instrument hinges and for use as sharpening oil. Physically harmless to DAB 10.

1 Turn off the back end by twisting the handle counterclockwise

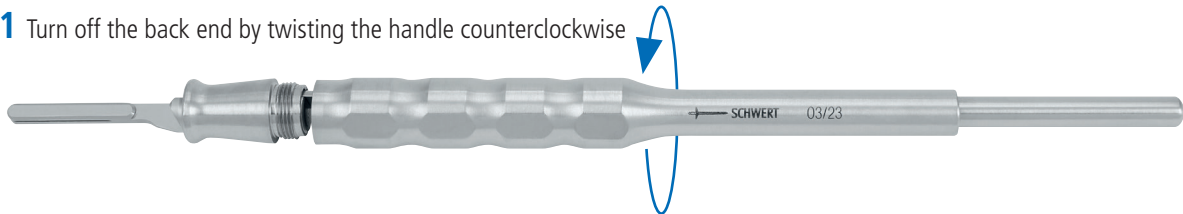

2 Remove the back end and the insert

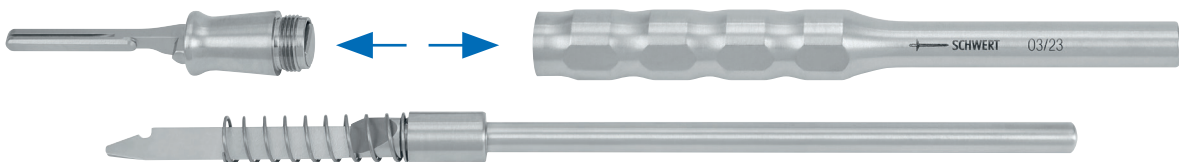

3 Oiling after cleaning with Medical White Oil

